


Breeze

by

ACCESS™



Breeze Cane holder installation

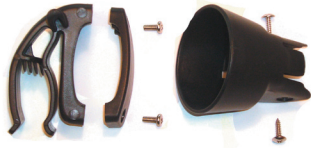

Caneholder kit


- Cane cup
- Fastening bracket
- Cane clamp
- 4 screws

Tool needed

- Screwdriver (PH2)



Press the Cane cup onto the rear wheel fork. The holes in cup and fork should align.

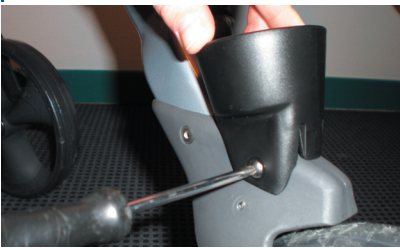

Fasten the cup with a screw on both sides



Place the cane clamp on desired height and fasten it to the bracket by screws.



Alternative

If used together with backrest. Let the backrest bow replace the fastening bracket. Screw the cane clamp directly on to the backrest bow.

Breeze Cane holder installation